

# **iHelp software**

(Android system v2.00)

## **User's Manual**

Made by: Xi'an Chinastar M&C LTD

Website: [http:// www.ihelpu.com.cn/english](http://www.ihelpu.com.cn/english)

Add: D Section Gazelle Valley, C-1 Region Pioneering Development Park #69 Jinye Road, Xi'an, Hi-tech Development Zone, Xi'an, Shaanxi Province, P. R. China

P.c: 710077

Tel: +86-29-88324785、 88311435

Fax: +86-29-88324785、 88311435

# Contents

1. Definitions .....	3
2. Download and setup .....	3
3. Operations.....	4
3.1 Add contacts .....	4
3.2 Bind .....	5
4. Functions .....	5
4.1 Message management.....	5
4.2 Call .....	5
4.3 Medicine reminder setting.....	6
4.4 Inquiring .....	6
4.5 Reminder setting.....	6
4.6 Broadcast setting .....	6
4.7 Remote power off.....	7
4.8 GPS setting.....	7
4.9 Unbind.....	7

www.technologysis-hts.gr

## 1. Definitions

Fall Warner named iHelp----- Fall Warner;

Phone number of guardian----- Binding phone number;

Message to Fall Warner----- Command message.

## 2. Download and setup

Users of the smart phone with android system download this App software, following the below methods:

- 1) Download from the website: [www.ihelpu.com.cn/english](http://www.ihelpu.com.cn/english);
- 2) Setup according to the enclosure CD;
- 3) Download and setup from the App store of Android.


### Attention:

**This App includes four parts(iHelp+GoogleMap+GooglePlayService+GooglePlayStore), user must make sure to setup all of those before using.**


www.technology-nts.gr

### 3. Operation


After setting up successfully, first click the software icon to run it, the operation interface as following:

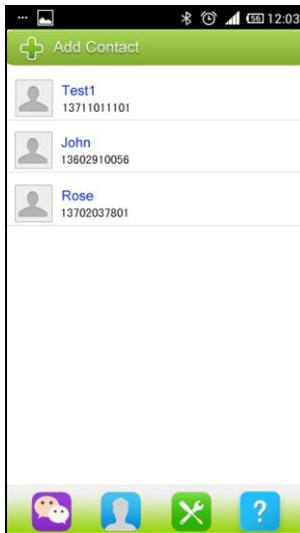


Interface 1  
(Add Fall Warner)

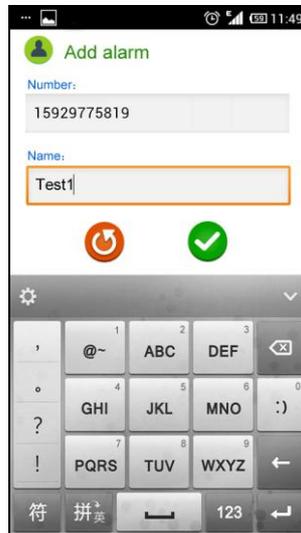


Interface 2  
(Add Name)

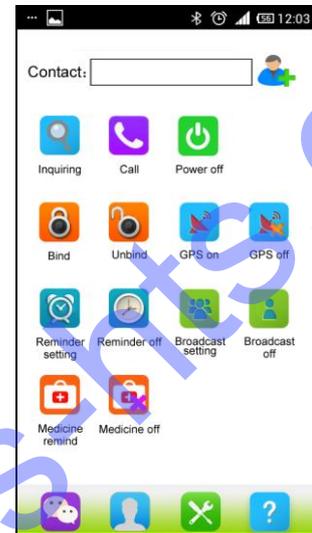

Interface 3  
(Commands)

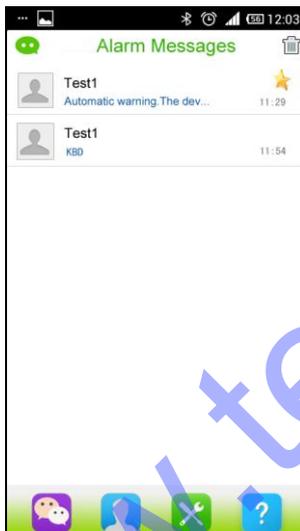

Interface 4  
(Alarm Messages)

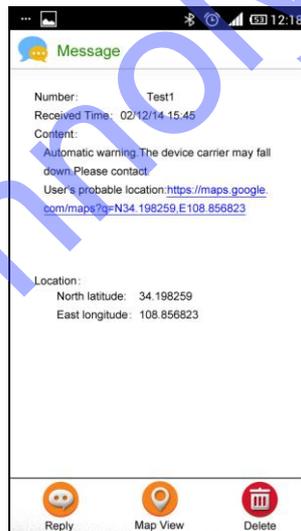

Interface 5  
(Message)



Interface 6  
(Broadcast Setting)

Before using iHelp, the software should be bound with Fall Warner to make sure the communication between Binding phone number and Fall Warner. Every Fall Warner can only bind with one phone number.

#### 3.1 Add contacts

After running the software, click the  button to turn into the interface 1, then click



on the top, inputting the phone number and name of Fall

Warner, click  to make sure this action.

User can click  to delete some information if there is some wrong inputting.

### 3.2 Bind

User clicks  in the bottom of the interface 3, and clicks  to choose one of the Fall Warner,

then clicks  to make sure this action.

User clicks  and then clicks  in the prompt box. About 10 to 15 seconds later, user will receive this message in the interface 4:

“Open binding is successful. Thank you for using our products. Email: ellen@web sensor.cn. Website: www.ihelpu.com.cn/english.”, it means the operation of binding is successful.

If user don't received this message, please click  to dial the number of Fall Warner,

When the Fall Warner rings, that means the Fall Warner has been ready to receive the message, please hang up this calling and re-click  as above.

**Attention: Please make sure that the card is GSM network card.**

## 4. Functions

User can do following operations after the binding is successful, if there are some wrong, user can click  to cancel.

### 4.1 Message management

User clicks  at the bottom of the interface 4 and can view all of the messages, such as “Automatic warning. The device carrier may fall down. User's probable location: <https://maps.google.com/maps?q=N3.198275,E108.856831>”, the location will be shown in the map after user clicks the link of the message.

User can click  to delete some messages in interface 4, user can choose some of them to delete or click “Delete All” delete all the messages.

### 4.2 Call

User clicks  to skip into the interface 3 and clicks  to choose one of the Fall Warner, and then clicks  to make sure this action, and then clicks  to dial the Fall Warner which you chose just now.

### 4.3 Medicine reminder setting

User clicks  to skip into the interface 3 and clicks  to jump into the interface of taking medicine reminder setting, then user can click  to set the time of medicine. It's 24 hours form and the left means hour, the right means second. User can set the time with “+” or “-” then click  to finish this action, the time of medicine reminder could be no more than 3 times everyday.

User also can click  to switch off the function of medicine reminder in the interface 3.

### 4.4 Inquiring

User can click  at the bottom of software to skip into interface 3, click  then choose the  in prompt box, user could inquire the current status of Fall Warner. The status message will be displayed in the alarm messages box. Click the  text button then could view all the alarm messages.

### 4.5 Reminder setting

User can click  at the bottom of software to skip into interface 3, and click  to set the reminding time, and the left means hour, the right means second. User can set the time with “+” or “-”, then click  to finish this action, it can be set only one time.

User also can click  to shut off the function of reminder in the interface 3.

### 4.6 Broadcast setting

User can click  at the bottom of software to skip into interface 3, click  to jump into the interface 6, there is a white box on the top, input the phone number in this box then click  to finish this adding action. The broadcast number should not be more than 5 recipients. After all, user clicks  to finish the broadcast setting.

User also can click  to shut off the function of broadcast in the interface 3.

After the broadcast setting, the functions of Fall Warner have been changed as following:

#### a) Warn automatically

While the device detected the falling action, Fall Warner will send warning message to binding phone number and broadcast numbers.

#### b) Automatic Call

while user makes a calling, Fall Warner will dial the Binding phone number ,if this number is blank or no answer, Fall Warner will dial the mass text numbers in sequence until making a communication, if all of those numbers are blank or no answer, the calling will be end.

### 4.7 Remote Power off

User can click  at the bottom of software to skip into interface 3, click  and , Fall Warner will be powered off with long-distance, and all functions are unavailable.

Recovering all the functions needs to be turn on the power of Fall Warner.

### 4.8 GPS setting

The GPS stays off in default and will be turned on when in the status of warning or inquiring. User can click  at the bottom of software to skip into interface 3, and click  to switch on the GPS and then click  to make sure this action, the GPS will be switched on. The Fall Warner will last for about 16 hours at this state.

User also can click  to switch off the GPS in the interface 3.

### 4.9 Unbind

User can click  at the bottom of software to skip into interface 3, and click  to unchain the binding between Fall Warner and Binding phone number, then click  to make sure this action. After that, all of the functions of Fall Warner will be restored to its factory settings.